

Pectoralis Implants

Silicone-gel-filled pectoralis implants available in anatomical shapes for the left and right pectoralis muscle



Left pectoralis



Right pectoralis

Implants made by POLYTECH – QUALITY made in Germany



Pectoralis Implants, CE 0483

Product Description

Silicone gel-filled pectoral implants are available as oval, smooth or micro-textured implants for augmentation and reconstruction to compensate soft tissue deficiencies.

Due to the higher stress that can possibly act on the implants, e.g. compared to mammary implants, a gel with firm consistency has been chosen. For easier insertion a tongue is fixed on the medial edge of the implant.

Pectoral implants are single-lumen implants, available in three different sizes and formed to fit the left or right side of the chest.

Information on Material

The shell of all pectoral implants consists, similar to that of gluteal implants, of a strong, resilient and highly elastic silicone elastomer equipped with a barrier layer. The implants are filled with a highly cross-linked silicone gel.

Production of the implants is based on a quality management according to ISO 9001 and ISO 13485. All materials used are of implantable medical quality.

Instructions for Use

Pectoral implants are used for aesthetic and reconstructive surgery. The approach normally is carried out via the axilla with the patient in supine position, the arm in 90°-abduction in order to elevate the muscle. The implants can be positioned submuscularly.

Indications

- Augmentation
- Contour correction
- Reconstruction, e.g. after trauma
- Poland Syndrome
- Implant exchange

Important Information

Pectoral implants are delivered individually packed and sterile.

Attention: Please note the detailed instructions for use included with each product.


Pectoralis implants

Silicone-gel filled, macro- or micro-textured surface, individually packed, sterile, CE 0483

Order n°		Volume (ml)	Dimensions (cm)		
POLYsmooth®	MESMO®		A	B	C
10500-190R	15500-190R ¹	192	14.7	10.2	2.4
10500-190L	15500-190L ¹	192	14.7	10.2	2.4
10500-230R	15500-230R ¹	231	16.0	10.7	2.7
10500-230L	15500-230L ¹	231	16.0	10.7	2.7
10500-300R ¹	15500-300R ¹	301	16.9	11.6	2.9
10500-300L ¹	15500-300L ¹	301	16.9	11.6	2.9

R = right; L = left

approximate measurements

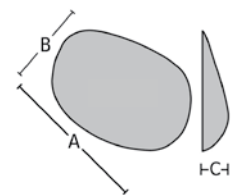


On stock 1 Supply after order ≈ 6 weeks

Incision



Undermining and positioning

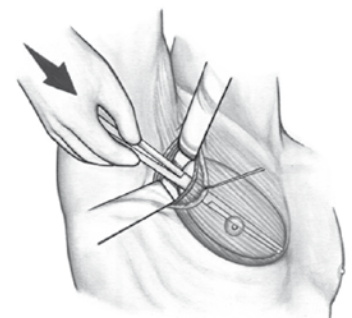

Final result

